Auto Mode per treated area

Preset(frequency, interval, time) Auto Mode will guaranty the maximum effect with convenient



Technical Specification

Name of Brand: Magsculpt	Type of Energy: Magnetic Field
Shape of Stimulation Pulse: Symmetrical Biphasic Wave	Magnetic Field Intensity: 3.1 Tesla
Power Input: 230V, 50/60Hz	Maximum Power Consumption: 2.5kVA
Purse Repetition Rate: 10~100Hz	Purse Duration: 400 µs
Dimensions(mm): W440*D600*H1160	Weight: 86kg









MAGSCULPT



MAGSCULPT

Everyone wants to have a slender figure with muscles and without unwanted fat. So many people want to try a device but are worried about heat, freezing, pain, surface damage and a long time commitment. Magsculpt will solve all these concerns.

Clients feel only

DEEP MUSCLE CONTRACTIONS

during operating like doing the most intense exercise!

No Heat

No Freezing

Magsculpt is


No Pain

No Surface Damage

No Down Time

The applicator generates **Electromagnetic pulses**, the client just lies down in a bed for 30min. for one area treatment only for 4sessions for 2weeks.

2 Tesla High Density Electro Magnetic Energy repeat contract and relax your muscles Max. 36,000times so deeply during 30min. operating time. It gives the equivalent effect of 36,000 times of doing sit-ups or squats.

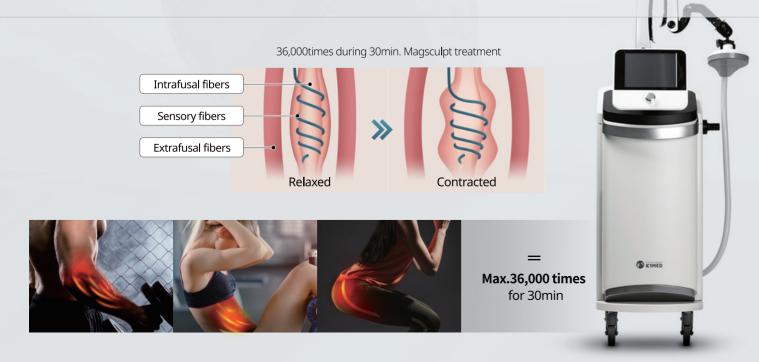


Do Magsculpt to Strengthen your core muscles and remove your stubborn fat

Thousands of repeated deep contractions of muscles rebuild firm muscle, it increases fat metabolism and leads to an increase in the rate of producing fatty acids. So fat cells are destroyed.



Magsculpt Mechanism for Muscle and FAT

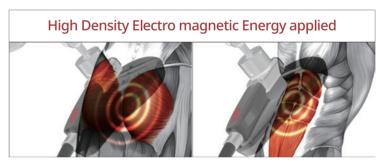

MUSCLES RESPONSE

- Rectus abdominis
- Transverse abdominis
- Obliques
- Gluteus muscle
- All skeletal muscles contract
- it gives hypertrophy(increase in volume) effect and hyperplasia(increase number of fibers).

BODY RESPONSE

Muscle Contraction

- Release Epinephrine > Break Down Fat
- Free Fatty Acid > Fat is used for energy

